BIOM® Optic Set HD Professional/HD Flex


für den Einmalgebrauch, für f=175 mm oder f=200 mm
disposable, for f=175 mm or f=200 mm
desechable para un solo uso, para f=175 mm ó f=200 mm
jetable, pour f=175 mm ou f=200 mm
monouso, per f=175 mm o f=200 mm
para uma utilização, para f=175 mm ou f=200 mm
Для одноразового пользования, для f=175 мм или f=200 мм

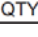Montageanleitung de, Seite 7
Assembly instructions en, page 12
Instrucciones de montaje es, página 17
Notice de montage fr, page 22
Istruzioni per il montaggio it, pagina 27
Instruções de montagem pt, página 32
Руководство по монтажу ru, стр. 37

Symbole / Symbols / Símbolos / Symboles / Simboli / Símbolos / Символы

	<p>Hersteller Manufacturer Fabricante Fabricant Produttore Fabricante Производитель</p>		<p>Herstellungsdatum Manufacturing date Fecha de fabricación Date de fabrication Data di produzione Data de fabrica�o Дата производства</p>
	<p>Nicht mit einem Messer �ffnen Do not open with a knife No utilice cuchillo para abrir Ne pas ouvrir avec un couteau Non aprire con il coltello N�o abra com uma faca! Не открывать ножом</p>		<p>Verwendbar bis Use by Fecha de caducidad � utiliser avant Utilizzabile fino al Utiliz�vel at� Использовать до</p>
	<p>Bei besch�digter Verpackung nicht verwenden Do not use when packaging is damaged No utilizar si el embalaje est� da�ado Ne pas utiliser si l'emballage est endommag� Non utilizzarlo se la confezione � danneggiata Com embalagem danificada n�o utilizar Не использовать в случае повреждения упаковки</p>		<p>Temperaturbegrenzung Lagerung Storage temperature limit Temperaturas l�mite de almacenamiento Temp�ratures de stockage admissibles Limite di temperatura per la tenuta in magazzino Limite de temperatura armazenamento Ограничение температуры хранения</p>
	<p>Relative Luftfeuchte / relative air humidity / humidit� relative/ umidit� atmosferica relativa / humedad relativa humidade do ar relativo относительная влажность</p>		<p>Luftdruck / air pressure / pression d'air / pressione atmosferica / presi�n atmosf�rica / press�o do ar / атмосферное давление</p>

	<p>Trocken aufbewahren Keep in a dry place Conservar seco Conserver au sec Luogo asciutto Guardar em lugar seco Сухое место / сухо сохраняют</p>		<p>einmal verwenden Use once De un solo uso. Utiliser une seule fois. Monouso Usar uma vez Использовать один раз</p>
	<p>steriles Produkt, einmal verwenden Sterile product, use once Producto estéril de un solo uso Produit stérilisé, à n'utiliser qu'une seule fois Prodotto sterile, usa e getta Produto esterilizado, usar uma vez Стерильное изделие одноразового применения</p>		<p>CE Kennzeichnung mit benannter Stelle CE marking with notified body Cód. de pedido (cód. de artículo) Marcado CE con organismo notificado Marquage CE avec identification de l'organisme notifié Marchio CE con l'organismo notificato Marcação CE com organismo notificado Маркировка CE с указанием уполномоченного органа</p>
	<p>Stückzahl Quantity Nº de unidades Quantité N. di pezzi... N.º de peças Количество</p>		<p>Sterilisiert mit Ethylenoxid Sterilized with ethylene oxide Esterilizado con óxido de etileno Stérilisé avec de l'oxyde d'éthylène Sterilizzato con ossido di etilene Esterilizado com óxido de etileno Стерилизация этиленоксидом</p>

	<p>Gebrauchsanweisung beachten Observe instruction sheet Siga las instrucciones de uso Observer la notice d'utilisation Osservare le Istruzioni per l'uso Observar as instruções de utilização Следуйте инструкции по эксплуатации</p>		<p>Achtung Attention Atención Attention Attenzione Atenção Внимание</p>
	<p>Enthält kein Naturkautschuklatex Does not contain natural rubber latex No contiene látex de caucho natural Ne contient pas de latex de caoutchouc naturel Non contiene lattice di gomma naturale Não contém látex de borracha natural Не содержит латекс из натурального каучука</p>		<p>Bestellnummer (Artikelnummer) Order number (article number) Cód. de pedido (cód. de artículo) N° de commande (n° d'article) Numero di ordinazione (numero dell'articolo) Número de encomenda (número do artigo) Номер заказа (номер артикула)</p>
	<p>Losnummer/Chargennummer Lot number/batch number Lote n° N° du lot/n° de la charge N. lotto/N. partita Número do lote/número da carga Номер партии/Серийный номер</p>		<p>USA: verschreibungspflichtig USA: Only available by prescription EE. UU.: con receta médica USA : uniquement sur ordonnance USA: da vendersi dietro presentazione di ricetta medica USA: receita obrigatória США: отпускается по рецепту</p>


**Kombination / combination / combinación / combinaison /
combinazione / combinação /КОМБИНАЦИЯ**

Objektiv / lens / objetivo / objectif / obiettivo / objetiva / ОБЪЕКТИВ	BIOM®	BIOM® Optic Set
f=175 mm	BIOM® 3m/c, 4m/c, 5m/c	HD Professional 54411
f=200 mm	BIOM® 3m/cl, 4m/cl, 5m/cl	HD Professional 54412
	BIOM® 3m/c, 4m/c, 5m/c	HD Flex 54415

**Umgebungsbedingungen / Ambient conditions / Condiciones ambi-
entales / Conditions aG/54411/0818/inmbiantes / Condizioni ambi-
entali / Condições do ambiente / Условия окружающей среды**

	Temperatur / temperature / temperatura / température / temperatura / temperatura / температура	Relative Luftfeuchte / relative air humidity / humidité relative/ umidità atmosferica relativa/ humedad relativa humidade do ar relativo относительная влажность	Luftdruck / air pressure / pression d'air / pressione atmosferica / presión atmosférica / pressão do ar / атмосферное давление
Transport / transport / transport / trasporto / transporte / transporte / перевоз	-10°C – +50°C (-14°F – +122°F)	10 % – 85 %	500 hPa – 1060 hPa
Lagerung / storage/ stockage / stoccaggio / almacenamiento / armazenagem / хранение	+5°C – +50°C (41°F – +122°F)	10 % – 95 %	700 hPa – 1060 hPa
Gebrauch / use / usage / uso / uso / uso употребление	+10°C – +40°C (+50°F – +104°F)	30 % – 90 %	800 hPa – 1060 hPa

**Übersicht / Overview / Vista general / Aperçu /
Visione d'insieme / Vista geral / Обзор**

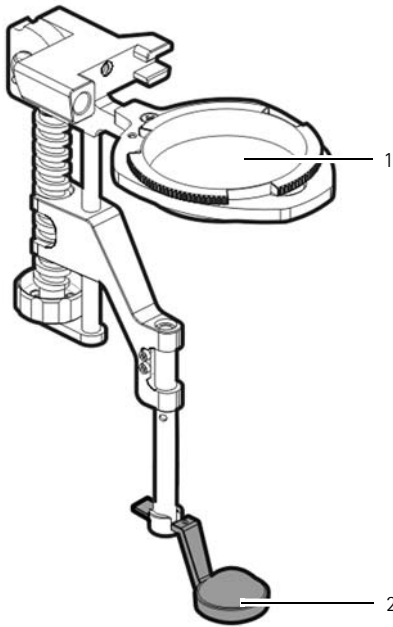1:
 Reduzierlinse
 Reduction lens
 Lente reductora
 Lentille de réduction
 Lente di riduzione
 Lupa frontal
 уменьшающая линза

2:
 Frontlupe
 BIOM® HD disposable lens (front lens)
 Lupa frontal
 Loupe frontale
 Lente d'ingrandimento frontale
 Lente reductora
 фронтальная лупа

**Technische Daten / Technical data / Datos técnicos /
Caractéristiques techniques / Dati tecnici / Dados técnicos /
Технические характеристики**

1	∅	53,5 mm (2.1 in)
	Höhe / Height / Altura / Hauteur / Altezza / Altura / Высота	10 mm (0.4 in)
	Gewicht / Weight / Peso / Poids / Peso / Peso / Вес	~ 8,1 g
2	Breite / Width / Anchura / Largeur / Larghezza / Largura / Ширина	18 mm (0.71 in)
	Tiefe / depth / hondura / hauteur / profondità / profundidade / глубина	45 mm (1.78)
	Höhe / Height / Altura / Hauteur / Altezza / Altura / Высота	21,3 mm (0.84 in)
	Gewicht / Weight / Peso / Poids / Peso / Peso / Вес	~ 2,5 g

**Konformität / Conformity / Conformidad / Conformité /
Conformità / Conformidade / Сертификат соответствия**

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Assembly instructions

The BIOM® Optic Set is an accessory of the OCULUS BIOM®. These instructions only describe assembly of the reduction lens and front lens. Further information, e.g. on use of the drive belts and the sterile cap, is included in the instruction sheet of the BIOM®.

Scope of delivery

Components	Order number
BIOM® Disposable Optic Set for the BIOM® consisting of BIOM® HD disposable lens (front lens), disposable reduction lens, 2 drive belts for motorized BIOM® and single use thumbscrew cover	
HD Professional for f=175 mm	54411
HD Professional for f=200 mm	54412
HD Flex for f=175 mm or f=200 mm	54415
Assembly instructions BIOM® Optic Set	G/54411/.../en

The manufacturer reserves the right to make changes in the scope of delivery due to further technical development.

- ➔ If you discover transport damage in your delivery, please notify the transporting company immediately.

Safety notes

- ➔ Please read the instruction sheet carefully.
- ➔ Only work with the BIOM® Optic Set when you have understood the instruction sheet.
- ➔ Take care to keep the instruction sheet near the BIOM® Optic Set.
- ➔ Carefully follow the instruction sheet of the BIOM®, in particular the safety notices.
- ➔ Never use a damaged BIOM® Optic Set.
- ➔ Use the BIOM® Optic Set only once.
- ➔ Ensure that the sterile packaging and the BIOM® Optic Set are not damaged.
- ➔ Never mount or dismount the BIOM® Optic Set above a patient.
- ➔ Observe the legal provisions valid in your country as well as the hygiene and disposal regulations of your hospital or clinic.

Proper use

The BIOM® Optic Set allows visualization of the ocular fundus as well as of vitreous body and retina structures during vitreo-retinal surgeries. It is intended for one-time use on the BIOM®.

→ Observe the safety notes listed above.

Storage

Type of packaging	Storage
Cardboard box	<ul style="list-style-type: none"> ■ Unopened or closed dust-tight ■ In a dark room ■ Ambient conditions: refer to page 4 ■ Storage duration: see "Use by" date on the package
Non-sterile outer packaging	<ul style="list-style-type: none"> ■ Protected storage (e.g. in cabinets or drawers): 6 months, but not longer than the expiration date ■ Unprotected storage (e.g. on shelves): Use within 48 hours
Sterile inner packaging	<ul style="list-style-type: none"> ■ For immediate use

→ Before taking a package out of an open box, clean all dust off the cardboard box.

→ Do **not** refill the box.

Unpacking

Caution

Patient hazard: Contamination caused by faulty unpacking
The BIOM® Optic Set is wrapped in sterile packaging. If the BIOM® Optic Set is unpacked inexpertly, it may become contaminated.

→ When unpacking, please take care that the BIOM® Optic Set remains sterile.

The BIOM® Optic Set is wrapped in dual packaging.

- Non-sterile outside packaging
- Sterile inside packaging

Non-sterile personnel:

- Check packaging for damage. If the packaging is damaged, the BIOM® Optic Set must not be used.
- Remove the outside packaging. Do not touch the inside packaging.

Sterile personnel

- Take the BIOM® Optic Set out of the sterile packaging.


Installation with sterile assistance

Caution


Patient hazard caused by incorrect installation

- Never mount the BIOM® Optic Set above a patient.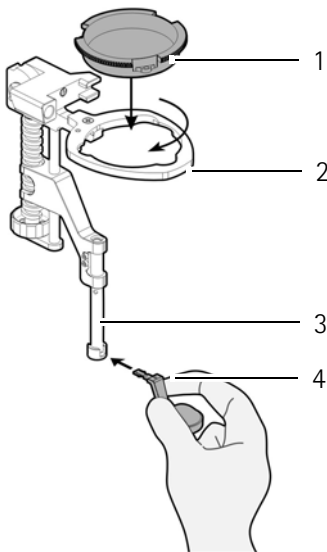

- Take the reduction lens and the front lens from the packaging without touching the surfaces of the magnifier.
- Hold the reduction lens (1) on its flange; insert it into the reduction holder (2) of the BIOM® so that the lugs of the socket are in the gaps of the receptacle.
- Rotate the reduction lens (1) clockwise until the resistance of the spring rest has been overcome.
- The reduction lens is then secured against falling.
- Now hold the front lens (4) with your thumb and index finger on the side of the lens holder.
- Push the plastic lug as far as possible into the slit of the BIOM® lens receptacle (3) in order to fasten the front lens. Ensure that you overcome the resistance of the arrester.

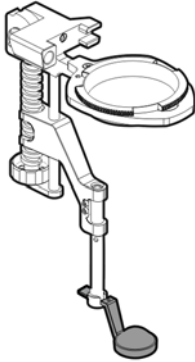The plastic spring hooks now engage the lens receptacle.

The lens is aligned correctly if the position is the same as in the drawing.

Note

The front lens may break off if handled incorrectly.

→ Insert the front lens correctly.

Dismantling and disposal


After use, the BIOM® Optic Set must be removed and disposed of..

Caution

Risk of personal injury caused by contaminated BIOM® Optic Set

- Never dismount the BIOM® Optic Set above a patient.
- After surgery, dispose of the disposable parts according to the hygiene regulations of your hospital and the legal requirements.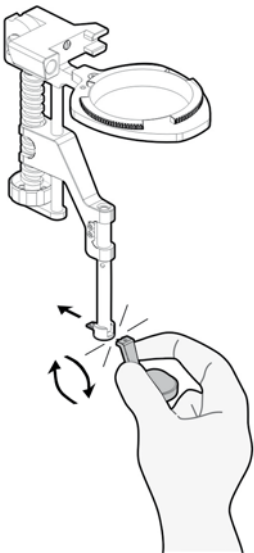

- Holding the front lens, bend this part of the holder swiftly up or down.
- Take the rear end of the plastic lug and pull it out to the rear.
- Turn the reduction lens counter-clockwise and take it out of the reduction lens holder.
- Dispose of the disposable parts according to the hygiene regulations of your hospital and the legal requirements.

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OCULUS Optikgeräte GmbH
OCULUS Surgical Inc.